


Labycare-Wex Bath. (SKSSWB347I)

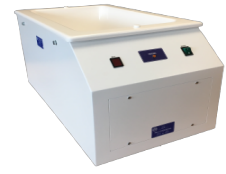

Wax bath therapy is intended to moisturize, soften the skin, improve blood circulation and reduce pain of joints. The wax bath uses paraffin wax that is soft and melts at a lower temperature than normal, therefore it does not cause any burns or blisters.

Description :

An established unit used around the world. This is efficient and odour-free and heats up to 18kg of wax. It is thermostatically controlled with a safety cut-out in

the event of lack of water. Wax bath thermometer T1A supplied as standard.

The Wax Bath comprises a resin tank which holds the paraffin wax, surrounded by an electrically heated, thermostatically controlled water jacket. The temperature control may be set to maximum in order to provide rapid melting of the solid wax when the unit is first switched on. After almost all of the wax is melted, the temperature control is normally set to approximately 50° C so that the paraffin is just in its molten state. It is advisable to check the actual wax temperature with a suitable thermometer prior to treatment.

Specifications :

Capacity: 12L

Working temperature: ambient to 70°C

Temperature accuracy: $\pm 1^{\circ}\text{C}$

Reservoir material: stainless steel

Reservoir size: ? 240mm x H260mm

Dispenser size: 360 x 430 x 460mm

Electrical: 230VAC 50Hz 1050W

labycare

www.labycare.com | Email: labycare@gmail.com